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BV-15

Fontos

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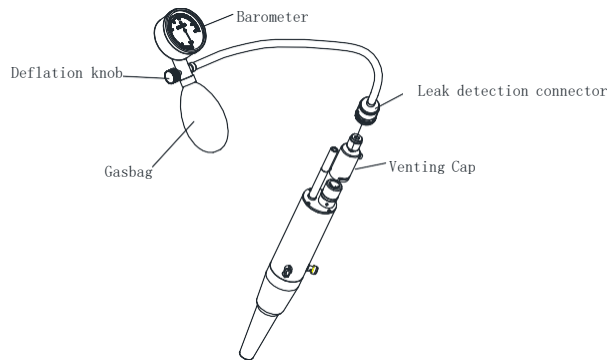
⚠ Caution

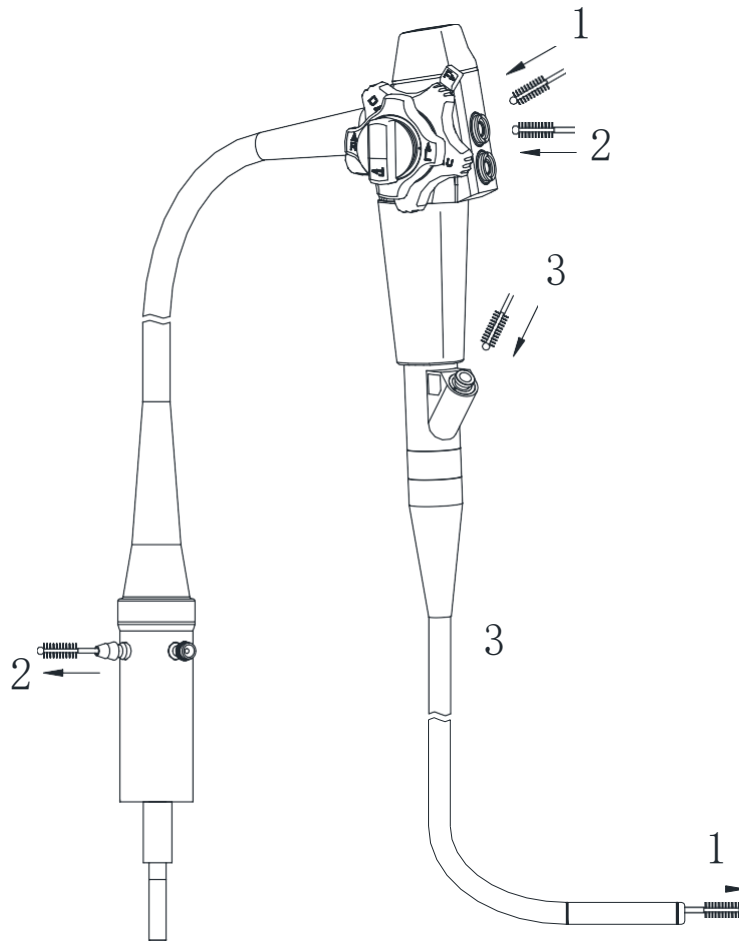
If the indicator needle moves, a leakage is indicated, remove the endoscope from the water, and contact us.

g. Clean the entire suction channel with cleaning brush

① Pass cleaning brush through suction valve housing at an angle of 45° until the brush extends from the distal end of the instrument. Brush channel several times.

② Pass the cleaning brush directly into the suction valve housing (approximately 90°)brushing the entire length of the universal cord until the brush extends from the suction connector. Brush several times.

③ Insert cleaning brush into channel opening approximately 10cm. Brush several times.

⚠ Caution
Please insert the cleaning brush in the direction indicated in the picture. Do not attempt to pass cleaning brush in reverse direction until the brush head extends from the distal end of the endoscope. Use only the cleaning brush to clean the channel.

h. Attach the biopsy seal Air/Water Valve and Suction Valve to the endoscope, and Repeat steps c, d and e again

i. Immerse the endoscope in disinfectant solution. Pump disinfectant solution through all channels. Allow the endoscope remain in disinfectant solution for recommended period of time.

Following disinfection, remove the instrument from disinfectant solution and place in clean water. Flush all channels with clean water until thoroughly rinsed. Then remove weighted end of intake tube from water and repeat flushing process, forcing air through channels to expel water. Flush all channels with 75% ethanol solution, forcing air through channels to expel

the ethanol solution.

Disconnect all channel irrigator and rinse the outside of the endoscope and all the button under running water.

j. Wipe the endoscope and all the buttons dry. Daub hospital grade silicone oil on the sealing ring of the button, connect the buttons to the endoscope.

k. Empty the water container after the final examination of the day, and disinfect the outside of it with 70% ethanol solution

3) Cleaning/Disinfecting/Sterilizing the Biopsy Forceps

The accessories should be thoroughly rinsing before disinfection/sterilizing.

The use of ultrasonic cleaner is desirable to aid in the removal of particulate matter. If possible, the accessories should be sterilized with ethylene oxide, and should be cleaned with gas to eliminate the poisonous gas. When the sterilizing can not be done, immerse the entire accessories into the disinfection solution. Wash and dry them thoroughly. To avoid sticking, lubricate the forceps cups with a medical grade silicone lubricant or liquid lubricant.

APPENDIX II TROUBLESHOOTING GUIDE

Refer to the listed methods. If the problems still can't be solved, stop using the instrument and send to us for repair.

	Sympton	Possible Cause	Solution
Image Quality	Image is not clear.	Dirty objective lens	Feed water to remove mucous, etc.
		There is bead or color list on image.	Stop using and return to Us for repair
Air/Water	Absent or insufficient Air/water feeding	Light guide connector is loose. Water bottle cap is loose.	Tighten the light guide connector. Tighten the water bottle cap.
		Air/water nozzle blocked	Soak tip of endoscope in warm soapy water. Or apply high-pressure air to eliminate fragment from the air pipe in light guide connector section.
	No air/water feeding	Air pump not turned on.	Turn air pump on.
Suction	Absent or insufficient suction	Suction valve blocked.	Remove valve. Clean the port with cotton swab. Lubricate with petroleum-based lubricants.
		Suction channel blocked.	Remove suction pipe. Depress the suction valve and apply more pressure to eliminate fragment from the pipe. .
		Semi-disposable biopsy valve is damaged	Replace with a new semi-disposable biopsy valve.
	Sticky suction valve	Suction Valve is dirty.	Remove and wash the valve. Clean the port by cotton swab with alcohol and lubricate with Petroleum based lubricants . Then reattach it.
	Fluid or air leak from the biopsy valve.	Valve deterioration or missing	Replace with a new valve.
Angulation	Tip deflection is abnormal.	Tip deflection is less than specifications.	Send back the instrument to Us for repair.
		There is interspace in angulation	(If resistance is encountered when rotating angulation control knobs, do not apply excessive force, otherwise early deterioration and damage will occur.
		Resistance is abnormal.	

		Angulation can not be locked	
		The outside of bending section is damaged	Stop using. Send back instrument for repair.
Accessory	Forceps do not operate smoothly	Forceps shaft is bent or kinked.	Discard and replace with new forceps
		Forceps cups dirty.	Soak in hot soapy water or hydrogen peroxide and brush to remove debris. The routine use of an ultrasonic cleaner to aid in cleaning the small sup hinges is recommended if problem does not persist (Do not clean the lens by ultrasonic cleaner)Lubricator forceps with silicone oil
	Forceps do not pass through channel smoothly	Forceps shaft is bent or kinked.	Discard and replace with new forceps.

APPENDIX III GUARANTEE AND SALES SERVICES

Certificate

Certificate card is added to this equipment.

Repair services after sale

1) Read the instruction manual again and inspect first when problems occurs.

2) Contact us if problems still can not be solved.

3) About the repair during the period of guarantee

Period of guarantee is one year, the starting time is the date promised in purchase contract.

The following damage or injury does not belong to the repair object.

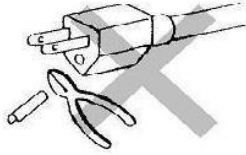
- Damage caused by natural disaster such as fire, gale, and flood etc.
- Damage resulting from improper use of the instrument or wrong operation.
- Products repaired or changed not by our company.

4) Cautions about repair

- Put the Video Processor which is to be repaired into the original package.

Please remember to write down the malfunction or damage of the instrument. And inform us your address, Zip code, operator's name and telephone number.


• Simple problems can be solved by qualified technician according to (Appendix II) Troubleshooting Guide. Any other repairment should be conducted only by us.

In case of current shock, all the light source and suction pump should be connected to the ground completely and securely.

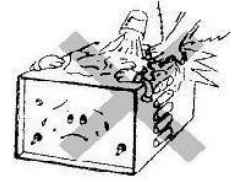
Direct sunlight, dust, high humidity and high temperature will damage instruments.


To prevent operator shock and damage to the equipment, keep spillable liquids away from electrical equipment.

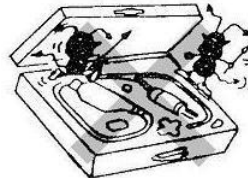
Do not bend insertion tube in a tight radius, as this will damage delicate fiber bundles.


Do not store in the carrying case.

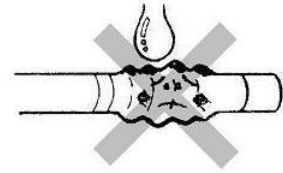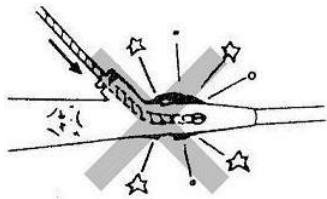
Petroleum based lubricants will cause stretching and deterioration of bending section rubber.

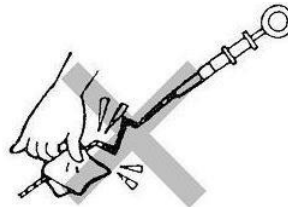
If forceps do not pass smoothly, do not force! Damage will occur to both biopsy forceps and instrument channel.

Do not apply excessive force.

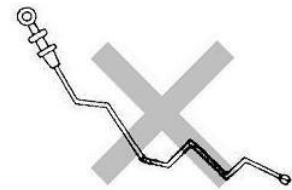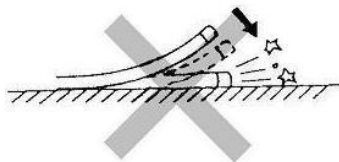
Any accessory which is kinked or bent along its shaft will not operate smoothly, will be difficult to pass through the video endoscope, and must be replaced.


Do not let distal end strike a hard surface as this may crack the objective lens.

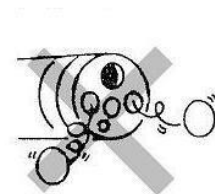Do not use a needle or other sharp object to remove debris from air/water nozzle. The nozzle may be deformed or pried loose.

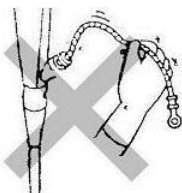Do not autoclave or boil the video endoscope, nor clean the video endoscope with ultrasonic cleaner.

To prevent bending or kinking forceps shaft, hold forceps close to biopsy valve and advance using repeated, short strokes.

Excessive air feed may cause discomfort to the patient, or cause damage to the stomach.