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邮箱：info@newtooo.com

First, thanks for using the Veterinary Anesthesia Ventilator VA-200, It is important that you carefully read all profiles from the randomized distribution before installing and using the product for the first time, which will help you to use the product better.

TooToo MediTech Co., Ltd . is committed to continuously improving product functions and improving service quality, and reserves the right to make changes to any product described in this manual and the content of this manual without prior notice.

Declaration:

TooToo reserves the right to modify the content of this user manual without prior notice.

Reserves the right to change the technology without prior notice.

Reserves the right to modify product specifications without prior notice.

Does not guarantee this material in any form, including (but not limited to) its implied merchantability and suitability guarantee for a specific purpose.

TooToo considers responsible for the safety, reliability and performance of the instrument only under the following circumstances:

The assembly operation, expansion, re-adjustment, improvement and repair are all carried out by personnel approved by TooToo;

The relevant electrical equipment complies with national standards;

The machine is used in accordance with the operating instructions.

If the following situations occur, TooToo is not responsible for the safety, reliability and operating conditions of the products:

The components are disassembled, stretched, and re-commissioned;

Repair or alter the equipment by non-Defy medical authorized personnel;

The product is not used correctly in accordance with the "User Manual".

1. Introduction

1.1 Summary

First of all, thanks for choosing the veterinary anesthesia ventilator produced by TooToo!

Before installing and using this product for the first time, please be sure to read all the materials shipped with it carefully, which will help you to use this product better.

TooToo is committed to continuously improving product functions and improving service quality, and reserves the right to make changes to any product described in this manual and the content of this manual without prior notice.

If you find that the actual situation of this product is inconsistent with this manual during use, or you want to get the latest information and any questions and ideas, please inquire or visit our company's website www.newtootoo.com.

This machine is only used for animal clinical and scientific research, and it is forbidden to be used for humans!

1.2 Safety

When the user operates the ventilator, it involves potentially dangerous substances that may exist. In order to avoid injury to lab animals and operators during use, as well as damage to the ventilator, please read carefully the "2-System Safety" chapter.

If you have any questions or suggestions on safety, please contact our company for after-sales service support.

The ventilator should be operated and managed by trained professionals!

1.3 Product feature:

- The machine has the characteristics of simple and generous appearance, elegant color matching, compact layout, and small overall size. It can be equipped with a anesthesia machine stand, which is convenient and flexible to operate;
- With machine-control mode and manual mode;
- Pressure limited working mode to ensure safety;
- Tidal volume setting, knob adjustment direction indicator;
- Equipped with two bellows of different sizes, 300ml and 1500ml, suitable for animals within 100kg;
- With AC power failure alarm and battery undervoltage alarm.
- Provides internal rechargeable battery power supply, full power can work normally for about 4 hours

1.4 Introduction to the main components of the ventilator

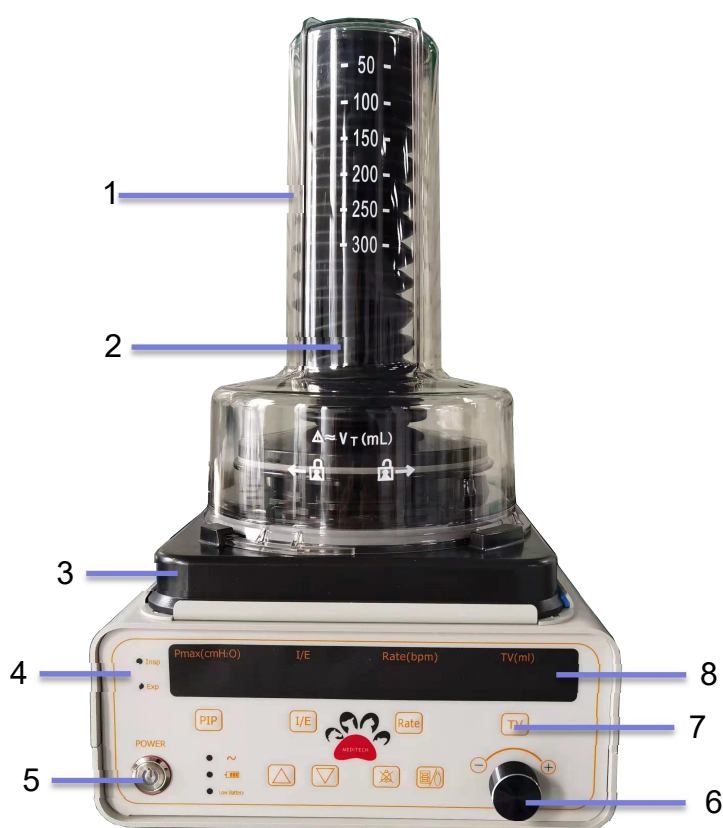
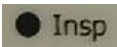
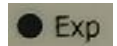
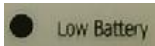
Figure 1 Front view of the ventilator

	Component	Description
1	Bellows components	Separate the outside gas from the driving gas
2	Folded bag	Deliver fresh oxygen and anesthetic gas to animal lungs
3	Bellow base	Connect the unit and the bellows
4	Indicator light	Indicate the operating status of the machine, a total of 7 indicator lights
5	Switch	Turn the device on or off, click to turn on, the power light is on, press again to turn off, the light is off
6	Tidal volume adjustment knob	Adjust the size of the inspiratory airflow of the ventilator, increasing clockwise and decreasing counterclockwise
7	Button	The parameters of the ventilator can be adjusted by pressing the buttons

	Component	Description
8	Digital display window	Display the setting parameters of the ventilator or monitor the tidal volume

Introduction of indicator lights

	Mark	Description
1		Inhalation indicator light, inspiratory state is on, exhalation state is off
2		Exhalation indicator light, the exhalation state is on, and the inhalation state is off
3		AC power indication, the AC power supply is intact, the indicator light is on; when the power is on, the indicator light flashes when the AC power is off
4		The battery status indicator is a two-color indicator, the open blue indicator light flashes; the charging blue indicator light is always on; the battery power supply green indicator light is on
5		When the battery is undervoltage (<8.4V), this indicator flashes
6		When setting tidal volume, if the monitored tidal volume exceeds the set value within a limited time, this indicator will flash, prompting you to turn the tidal volume knob counterclockwise to reduce the tidal volume
7		When setting tidal volume, if the monitored tidal volume does not reach the set value within a limited time, this indicator will flash, prompting you to turn the tidal volume knob clockwise to increase the tidal volume

Introduction of buttons

	Mark	Description
1		Inspiratory pressure limiting button
2		Inhale and exhale ratio buttons
3		Breath rate button
4		Tidal volume button
5		Increase parameter button
6		Decrease parameter key
7		Mute button
8		Machine-control, manual switch button

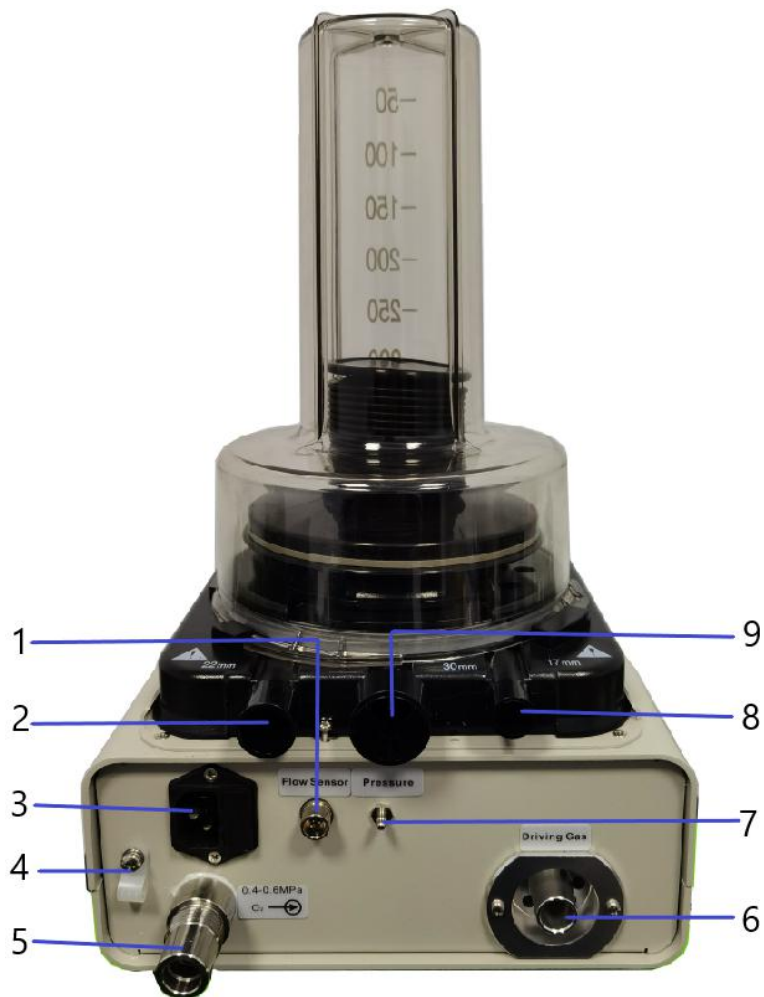
Figure 2 Back view of the ventilator

	Component	Description
1	Flow sensor interface	Connect the turbine flow sensor to measure the tidal volume
2	Fresh gas interface	Connect to the anesthesia machine
3	Power interface	Connect to AC power
4	Wire fixing slot	Fix the power cord to prevent it from falling off
5	Gas source interface	Oxygen gas source (or air)
6	Drive gas interface	Connect to 17mm interface

	Component	Description
7	Pressure sensor interface	Connect to 22mm interface
8	17mm interface	Connect to the drive gas interface
9	Anesthetic waste gas exhaust interface	Connect to the gas filter tank

1.5 Specifications

Parameter item	Specification
size	253mm×224mm×112mm
weight	3.22kg
Breath rate setting	2~120 bpm
Inhalation and expiration ratio setting	3:1, 2: 1, 1.5: 1, 1: 1, 1: 1.5, 1: 2, 1: 3
Inspiratory tidal volume setting	100~1500ml
Exhalation Tidal volume monitoring	10~1500ml
Airway pressure peak setting	5~60 cmH ₂ O
Airway pressure monitoring	0~99 cmH ₂ O
Animal weight range	≤100.0kg

2. System safety

2.1 Safety instructions

For safety, please pay attention to the following:

- Please read this user manual carefully.
- This user manual is an integral part of the equipment, and it is easy to refer to at any time when placed near the equipment.
- Ventilator working conditions:
Temperature: 0℃~40℃,

Humidity: 15%~95% (Non-condensing) ,
Atmospheric pressure: 70kPa ~ 110kPa
70kPa ~ 110kPa

Warning:

When high-pressure oxygen meets combustible substances (grease, oil, alcohol, etc.), it may cause an explosive reaction:

- Keep the equipment and all joints absolutely free of oil and grease
- Before operating the oxygen supply device, be sure to keep your hands clean.
- It is absolutely forbidden to smoke and light open flames near the ventilator.
- When installing and replacing the oxygen cylinder, use your hands as much as possible to tighten all the joints on the oxygen cylinder and the pressure reducing valve. If you need to use tools, do not use excessive force to prevent damage to the threads and sealing materials, which may cause gas leakage.
- Please take measures to prevent the oxygen cylinder from tipping over. If the oxygen cylinder falls on the pressure reducing valve, it may be crushed and cause an explosion.

Attention:

- Always open the oxygen cylinder pressure reducing valve slowly to prevent the surge of pressure from impacting the valve fittings inside the ventilator.
- Always check the oxygen pressure of the oxygen cylinder, do not use up the oxygen completely, otherwise the humid air in the surrounding environment may invade and cause the parts to rust.

Attention:

- The animals and ventilator must be continuously observed during breathing.
- The breathing circuit connected to the anesthesia machine must be unobstructed and easy to observe, and avoid operations that are covered or that may affect its function (for example, the position of the animal is improper).
- A large number of quality assurance measures have been taken when developing equipment software, and the risks caused by software defects are minimal.
- Please take measures to prevent the rubber and rubber parts from being exposed to ultraviolet light and long-term direct sunlight to avoid embrittlement of these parts.

2.2 Warning information and precautions

- Before using this ventilator to operate animals, users must read this user manual carefully and fully understand its content.
- Before using this ventilator, please check all interfaces carefully to ensure that the air tightness is good and there is no leakage.

- The user shall be responsible for the equipment failure caused by any improper cleaning, maintenance and operation of the equipment.
- If disassemble the main unit of the device without authorization from TooToo, the company will no longer provide any quality assurance and technical maintenance service commitments for the device. If you have any technical problems, please contact authorized personnel or TooToo for support.

 **Risk of electric shock:** Non-professionals are forbidden to remove the cover or panel of the device and disassemble the built-in rechargeable battery.

 **Danger:** It is forbidden to use flammable and explosive gas as the equipment gas source.

2.3 Safety notice

This machine is limited to personnel who have knowledge about respiratory anesthesia or who have been trained by the manufacturer.

The equipment should be installed in a place far away from water sources, well ventilated, and relatively constant air pressure, temperature, and humidity, avoid direct sunlight, and avoid corrosive, flammable and explosive gases and chemicals in the place. During placement or transportation, avoid tilting, vibrating, or squeezing the equipment! Pay attention to the voltage, current and frequency of the power supply, and pay attention to whether the grounding terminal is normal. Avoid running the equipment under overload conditions

Warning!

It is forbidden to use the rapid oxygenation button on the anesthesia machine during the inhalation phase of the breathing cycle. It can cause severe rupture of the animal's lungs. In the inhalation phase, the drain valve in the control unit is closed, so the oxygen-filled flow will be added to the aspiration flow of the ventilator, and the only destination of the flow is the lungs of the experimental animal.

2.4 Warranty Information

- The warranty of this equipment starts from the date of leaving the factory. the equipment cannot be used normally due to defects in materials and workmanship. The company is responsible for after-sales service such as equipment maintenance and component replacement.
- Any damage to the equipment caused by incorrect use or over-range use is not covered by the warranty. If repairs or parts replacement are required, the costs incurred shall be borne by the user.
- If the repaired equipment arrives, it is found after inspection that it has been disassembled without the authorization of TooToo, the company does not provide after-sales service such as warranty, free maintenance and parts replacement.
- The warranty statement (including its restrictions) is exclusively issued by TooToo, covering all other warranty conditions.

3. Unpacking and assembly

- 1) Before unpacking, please check the outer packaging carefully. If you find any damage, bumps, etc., please contact the supplier as soon as possible. After confirmation, you can proceed to the next step;
- 2) Take all the goods out of the shipping box, and properly keep the original packaging for future transportation;
- 3) Please count the materials one by one according to the packing list details (below), check whether the delivered materials and quantity are consistent with the contract order, and carefully check the materials. If there is damage, bumps, etc., please contact the supplier as soon as possible.

Packing list details:

Main Unit ----- 1

300ml bellow and its cover ----- 1

1500 ml bellow and its cover ----- 1

0.8 m 22mm pipeline ----- 1

(Purpose: Used to connect anesthesia machine and ventilator,can use steam sterilization)

0.5m 17mmReusable threaded tube ----- 1

(Purpose: Used to connect the air box and the ventilator, and can be steam sterilized)

airway pressure connecting tube-22mm -----1

Turbine flow sensor ----- 1

power cable ----- 1

Warranty card ----- 1

User Manual ----- 1

Attention : If your order contains other small accessories, they may also be packed and shipped at the same time, please refer to the freight bill.

3.1 Usage notice

Before using this equipment, please read this user manual carefully!

This equipment is designed to perform the functions mentioned in this manual. please use and operate according to the corresponding instructions in the manual.

This equipment must be regularly inspected and maintained to ensure the reliable operation of the equipment.

If any component has the following conditions: partial or overall damage, excessive wear, contamination or other reasons, which lead to the end of its service life, it should stop using it and replace it with accessories produced by our company or an enterprise authorized by our company.

Equipment that cannot work normally should be stopped immediately, and users are not allowed to modify the equipment in any form. If you do not follow the maintenance instructions authorized by the company, or perform unauthorized repairs or improvements on the equipment and its accessories, or use damaged accessories or unauthorized accessories and equipment, causing injury or Any damage to property (including the equipment itself) shall be borne by the user.

3.2 Installing

Check whether the control unit is in good condition, and check all interfaces on the rear panel to ensure that the air supply port (SUPPLY GAS), driving gas port (DRIVING GAS) and exhaust port (EXHAUST) are not blocked;

Check whether the bellows and bellows cover are intact, and check that there is no sundries inside the bellows;

Install the bellows and bellows cover. Put the bottom corrugated sleeve of the bellows on the assembly ring of the control host, gently hold the outer edge of the bellows disc (top of the bellows), quickly lift and put it down, repeat several times to smooth out the ripples on the bellows. Then put the bellows cover on the bellows and press it down gently, and turn the bellows cover clockwise until it is correctly locked in the bayonet of the base. At this point, the installation of the bellows and the bellows cover is completed.

Figure 3

Connect the turbine flow sensor. Insert the plug of the turbine flow sensor into the flow sensor interface (FlowSensor), as shown in Figure 4.

Figure 4

Install the breathing circuit port. Connect the airway pressure connecting tube-the airway pressure connecting tube (Luer connector) beside the Luer-22mm connector to the pressure sensor port (Pressure) of the control unit, and then install the 22mm connector (transparent) on the 22mm of the bellows base On the port; as shown in

Figure 5 below.

Figure 5

Connect the driving gas pipeline. Use a rubber booster tube to connect the 17mm port of the bellows base to the driving gas port on the control unit; as shown in Figure 6 below.

Figure 6

Connect the gas supply system. Connect the oxygen cylinder pressure relief valve (gas output pressure must be between 0.2~0.4MPa) and the gas source port (SUPPLY GAS) on the control host with an oxygen pipe and a three-way valve, insert it first, and then turn it clockwise to the end. As shown in Figure 7 below.

Figure 7

Power cord connection: First insert the end of the power cord into the power interface of the machine, and then use the cable fixing card to fix the power cord on the machine to prevent it from falling off. Insert the other end of the power cord into the power socket. As shown in Figure 8 below.

Figure 8

Connect the ventilator and anesthesia system. Remove the breathing bag from the anesthesia machine, and connect the 22mm port on the base of the bellows with the breathing bag interface of the anesthesia machine with a threaded tube; as shown in Figure 9 below.

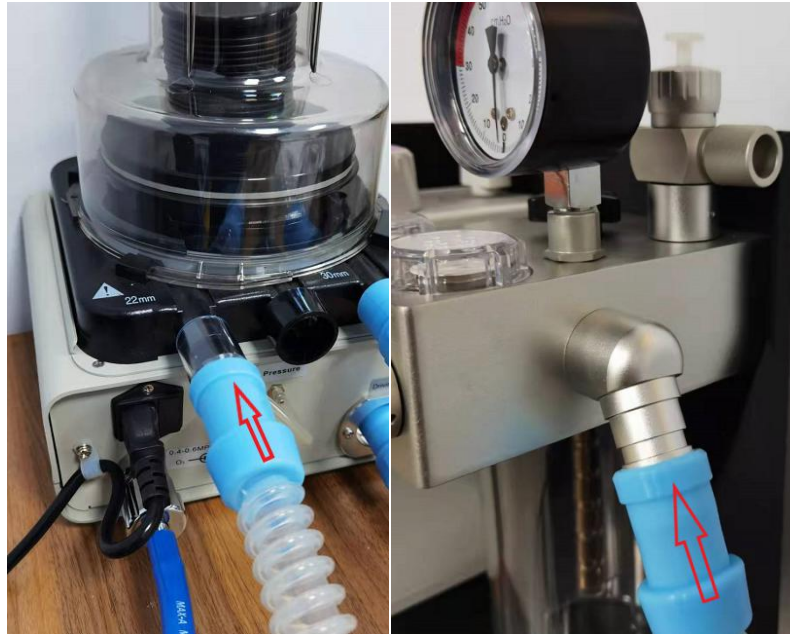
Figure 9

Install turbine flow sensor.

First, determine the minimum tidal volume. If it is greater than 100mL, it is recommended to install the turbo flow sensor on the inspiratory line, as shown on the left side of Figure 10; if it is less than 100ml, install it on the expiratory line, as shown on the right of Figure 10 below.

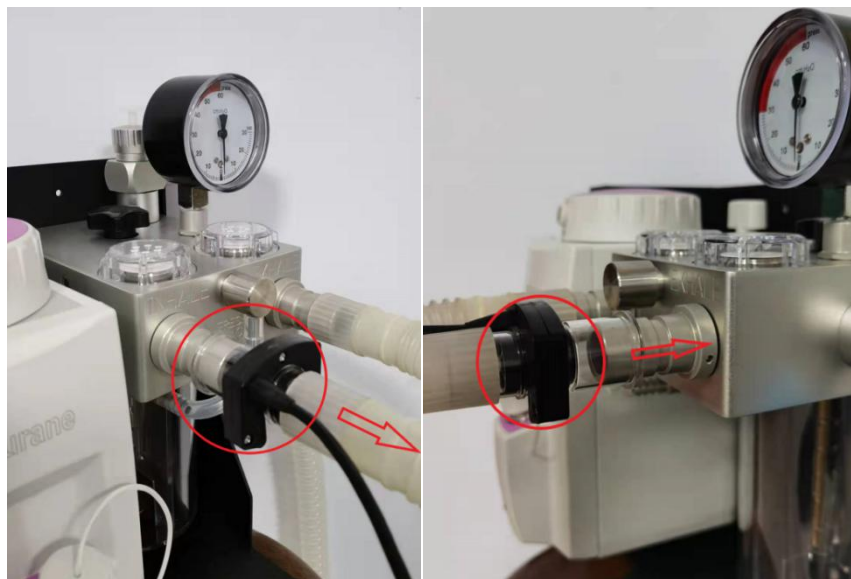
Figure 10

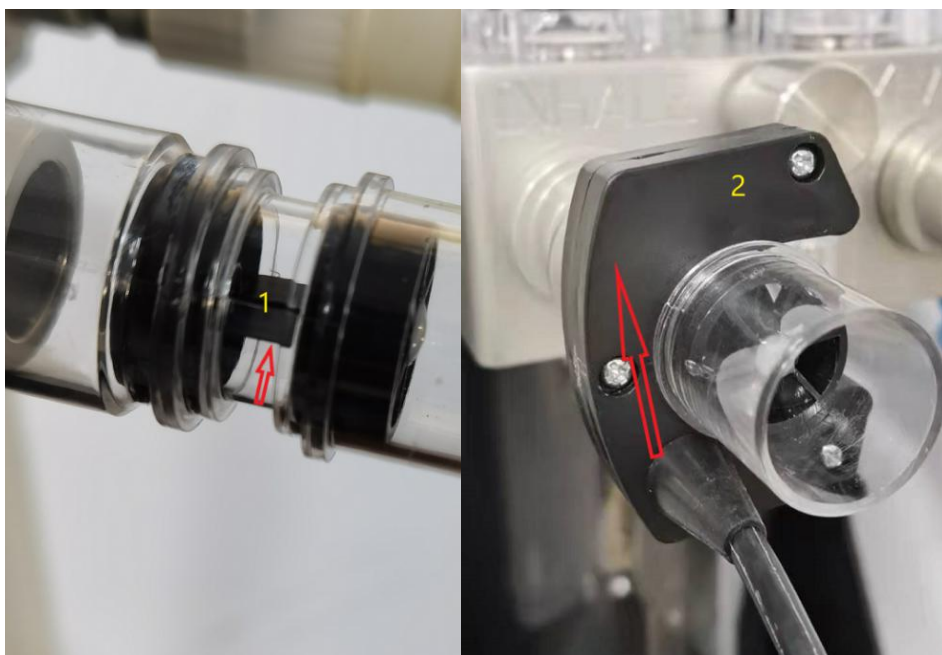
Figure 11

⚠ Attention: As shown in Figure 11 above, the turbine sensor consists of two parts, the left side shows the 1 turbine part and the right side shows the 2 detector part. After the turbine is stopped during installation, the turbine blade is in a free and stationary state. When installing the detector part, ensure that the direction of the detector (arrow direction in the right picture) and the direction of the turbine blade (arrow direction in the left picture) are perpendicular to each other. In order to reduce the disturbance and ensure the correct measurement results.

4. Operation guide

4.1 Turn on

Gently press the power switch (not too hard, it is easy to damage the power switch), the machine is powered on, the digital display window displays "8", all the indicators are on, and it enters the normal working state after 3 seconds, the digital window displays the set parameters, breathe The machine starts to work.

4.2 Parameter adjustment

Press the button that needs to adjust the parameter, the digital window of the corresponding parameter flashes, then press "" to increase the parameter, press "" to decrease the parameter, if you keep pressing "" or "", the parameter will increase or decrease rapidly After reaching the required value, press the button to adjust the parameter again, the digital window stops flashing, and the machine starts to work according to the new parameter.



4.3 Mode switch

Press "" to switch between machine control mode and manual mode. In machine control mode, the "IE" window normally displays the respiratory ratio parameters. In manual mode, the "IE" window displays "C C C C".

Attention: The turbine flow sensor can be installed on the inspiratory or expiratory line. When installed on the inspiratory line, the tidal volume needs to be adjusted to more than 100ml; when installed on the expiratory line, the tidal volume needs to be adjusted below 100ml, the tidal volume window "CCCC" is displayed. This must be done well, otherwise it will cause errors in the tidal volume monitoring of the ventilator.

4.4 Alarm and mute operation

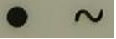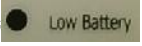When the AC power is suddenly cut off or the battery is under-voltage, the machine will sound an alarm. When the AC power fails, the AC indicator " " will flash; when the battery is under-voltage, the battery under-voltage indicator " " will flash.

When the machine emits an alarm sound, press the " " key to mute the machine. If the AC power failure alarm, the machine will not sound the alarm again, unless the AC power is re-powered and then the power is off again; if the battery is under-voltage alarm, it will be activated after 2 minutes Sound the audible alarm again.

4.5 Work status description

In the machine control mode, the frequency and inspiratory-expiratory ratio set by the machine are delivered according to the set parameters. However, when the airway pressure reaches the set "PIP" pressure, The machine will immediately turn to exhale for pressure protection.

If the turbine flow sensor is installed in the inspiratory pipeline, when the inspiratory tidal volume reaches the set tidal volume, the inhalation will be stopped and the inhalation will be switched to exhalation. If the inspiratory time is less than 80% of the set inspiratory time, the indicator light " " will flash, increase and decrease the inspiratory flow rate;

If the set tidal volume is not reached after 120% of the set inhalation time, the machine will stop inhaling and turn to exhalation, and the indicator light " " will flash, prompting to increase the inspiratory flow rate.

"PIP" window: Inhalation status, the display content of the window will dynamically increase with the increase of airway pressure. After reaching the peak pressure, the peak pressure will be displayed for 0.5 seconds, then the pressure will drop, and finally the end-expiratory pressure will be displayed.

"TV" window: If the turbine flow sensor is installed in the inhalation line, the inhaled tidal volume will be displayed during inhalation, and the maximum inhaled tidal volume will be displayed; if the turbo flow sensor is installed in the expiratory line, it will display the exhaled volume during exhalation. The tidal volume is displayed, and the maximum expiratory tidal volume is displayed.

5. Clean and maintain

5.1 Product cleaning

Most of the ventilator is not in direct contact with breathing gas, therefore, it only

needs to be cleaned with a dry cloth. Cleaning the surface of the ventilator: The surface of the ventilator can be wiped with a clean, soft, slightly damp cloth. You can use a neutral cleaning agent to remove stubborn dirt and stains.

Warning: Only use water and neutral detergent to clean the bellows and bellows cover, use a soft cloth to wipe, not rough cloth or smelly detergent. Be careful not to use ethanol for cleaning. Cleaning of the bellows cover: Rotate the bellows cover counterclockwise until the buckle is completely exposed (due to the action of the sealing ring, this step requires a large external force), and lift the bellows cover in the vertical direction. Since the bellows cover does not directly contact the breathing gas, it only needs to be wiped with a clean slightly damp cloth, or soaked with a neutral detergent and rinsed with water. High-temperature steam cannot be used to sterilize the bellows cover. The high-temperature steam will deform the bellows cover and cannot be used normally. Do not use ethanol for cleaning.

5.2 Battery maintenance

5.2.1 Maintenance of battery performance

During battery use, it should be optimized regularly to maintain its service life. It is recommended to maintain the battery every 3 months of storage. Please follow the steps below

- 1) Take out the equipment and stabilize it for more than 2 hours at a room temperature of 5°C~30°C;
- 2) Connect the external power supply and charge the battery for 6~8 hours.
- 3) Reinstall the equipment.

5.2.2 Check battery performance

The performance of the battery may decrease as the usage time increases. Under normal circumstances, the battery life when fully charged is 4H. If the battery power supply time is significantly less than 4H, consider replacing the battery or contacting after-sales personnel.

Note: The service life of the battery depends on the frequency and time of use. If the battery is properly maintained and stored, the life of the lithium battery is about 3 years. If the battery is used improperly, its life may be shorter. We recommend replacing the lithium battery every 3 years.

5.3 Battery recycling

If the battery is obviously damaged or the battery capacity is exhausted, it should be replaced and recycled correctly. When disposing of used batteries, follow the corresponding regulations.

Do not disassemble the battery, put it into fire, or short-circuit it. Burning, explosion, or leakage of the battery may cause personal injury.

6. Warranty

The warranty only applies to new products purchased directly from TOOTOO Company or from TooToo authorized distributors, and the first purchaser extended from it.

For a period of 1 year from the date of delivery of the first buyer or buyer's order, the original manufacturer will guarantee the functional defects of the anesthesia machine

in terms of materials and workmanship, and comply with the operating instructions, maintenance manuals, corresponding labels and In the product description of the insert, during normal use under the same operating conditions, regular maintenance, replacement and repair are carried out in accordance with the instructions.

If the product is repaired or serviced by an organization other than TooToo, or is inconsistent with the instructions provided by TooToo, or is changed by an organization other than TooToo, or the product is abused or misused and caused by problems , TooToo will not accept any warranty application mentioned above. TooToo is not responsible for any accidental damage, indirect damage, special operation and other damage caused by improper use of the instrument.

7. Product Service

If you have any questions, please keep in touch with us

After-sales service unit:

TOOTOO MEDITECH CO., LTD.

Address: RM 607, HAIHANG CITY BUSINESS CENTER, LONGGANG, SHENZHEN, CHINA.

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网址：www.newtootoo.com

TOOTOO MEDITECH CO., LTD.

Address: RM 607, HAIHANG CITY BUSINESS CENTER,
LONGGANG, SHENZHEN, CHINA.

Web: www.newtootoo.com